Protective concentrated gel for oral mucositis relief

Product Description

Gelclair® is a viscous gel specially formulated to aid in rapidly soothing the pain of oral mucositis. Gelclair® works by forming a protective film barrier that adheres to the mucosa of the mouth to protect the nerve endings that cause pain from further irritation and by hydrating and coating damaged tissue. In clinical studies, Gelclair® effectively treated mouth pain and provided rapid relief that helped patients eat and drink more easily.

Gelclair® is available as a concentrated gel for dilution with water.

Indications for Use

Gelclair® helps with the management of painful symptoms of mucositis of the oropharyngeal cavity. When used as a mouthwash, Gelclair® forms a protective film that helps to provide pain relief caused by medication, disease, radiotherapy, chemotherapy, oral surgery, traumatic ulcers caused by dental braces and dentures, diffuse aphthous ulcers, and ageing.

Gelclair® should be used 3 times a day 30 – 60 minutes before meals, or as recommended by your health care professional.

Ingredients

Polyvinylpyrrolidone (9%), sodium hyaluronate (0.1%), aqua, maltodextrin, propylene glycol, peg-40 hydrogenated castor oil, hydroxy-ethyl-cellulose, aroma, disodium EDTA, sodium saccharin, glycyrrhetic acid, and preservatives (potassium sorbate, sodium benzoate, benzalkonium chloride).

Precautions and Warnings

There are no known interactions with medicinal or natural health products. Do not use Gelclair if you have a known history of hypersensitivity to any of its ingredients.

Gelclair® may reduce the absorption of sublingual drugs. Consult your health care professional if you use any sublingual drugs. If swallowed accidentally, no adverse effects are anticipated.

Gelclair® may be used during pregnancy and lactation.

Gelclair® does not affect the ability to drive or use machines.

Do not swallow. Consult your health care professional if no improvement is seen after 7 days of treatment.

Keep out of reach of children.

Undesirable / Adverse Effects

No serious adverse events have been reported to date.

Non-serious reversible adverse events such as stinging, burning sensation, and local discomfort are reported rarely.

Accidental Ingestion / Overdose

There has not been any report of overdose with Gelclair®. Ingestion of Gelclair® is not recommended.

Storage

Store at room temperature (15°C – 30°C) away from direct sunlight. Do not freeze.

The gel may become a little darker and thicker during storage; however, this does not affect its efficacy or safety.

Do not use if the sachet is damaged.

Instructions for Use:

Squeeze the entire contents of the single-dose Gelclair® sachet into a glass.

Add 3 tablespoons of water (40 mL). Stir mixture well and use immediately;

Rinse around the mouth for at least 1 minute to thoroughly coat the tongue, palate, throat, inside of cheeks, and all oral tissue. Do not swallow;

If you are not able to rinse and gargle (e.g., in young children), you may apply the gel directly on the mucosal lesions of your mouth by using a sponge or swab;

Gargle and spit out. Discard any unused gel or mixture;

Wait 30 - 60 minutes before eating or drinking.

For best results, use Gelclair® three times a day, before meals or as recommended by your health care professional.

Manufactured by:

Biokosmes S.r.l. Via dei Livelli, 1, 23842, Bosisio Parini (LC), Italy

Manufactured for:

BioSyent Pharma Inc., 402-2476 Argentia Rd., Mississauga, ON L5N 6M1 Canada
1.888.439.0013 | www.gelclair.ca

Under license of Venture Life Ltd, Bracknell, United Kingdom

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Bioskosmes S.r.l. Via dei Livelli, 1, 23842, Bosisio Parini (LC), Italie
BioSynt Pharma Inc., 402-2476 Argentia Rd., Mississauga, ON L5N 6M1 Canada
Sous licence de Venture Life Ltd, Bracknell, Royaume-Uni